

Console Person



Equipment to bring

- ECMO console

Role

- Prepare ECMO circuit - NO heater
- Take over equipment role from C3

Procedure

Console Checks

1. Turn console on
2. RPM to 0
3. Unscrew yellow cap from oxygenator
4. Unclamp circuit and blue & red tubes
5. Run 2000 RPM for 2min (ensure flow in priming bag)
6. Reapply yellow cap
7. Clamp circuit (and both blue & red tubes close to de-airing bag)
8. RPM to 0, ensure ECMO clamp on
9. Apply white U/S cream
10. Zero flow on console
11. Connect oxygen to O₂ cylinder (not blender) at **3L/min**
12. Await for transfer to bedside
13. Ask C1 or C2 to check no air in circuit

Equipment role

1. Take over equipment role once C3 scrubbed

U/S

C2

C1

C3

Trolley

ECMO
Console

**Console
Person**

Team
Leader